

DUAL-SPECTRUM DPL PLATFORM

LUMICLEAR

Focused light. Expanded treatment versatility.

530-750 nm / 640-750 nm

2

FILTERED SPECTRUMS

1-10 Hz

ADJUSTABLE FREQUENCY

10 × 50 mm

SAPPHIRE WINDOW

FOCUSED VERSATILITY

Two light bands. One adaptable platform.

LumiClear brings narrow-spectrum DPL technology, dual treatment modes and contact cooling into one provider-focused system.

01 Dual-spectrum flexibility

Separate 530-750 nm and 640-750 nm filters support a broader range of professional light-based protocols.

02 Two treatment modes


In-Motion and Stamp modes give trained providers flexibility across treatment areas and protocol goals.

03 Efficient coverage

A 10 × 50 mm sapphire contact window supports consistent coverage across larger zones.

04 Integrated cooling

Semiconductor, air and water cooling are paired with contact gel to support a controlled treatment workflow.

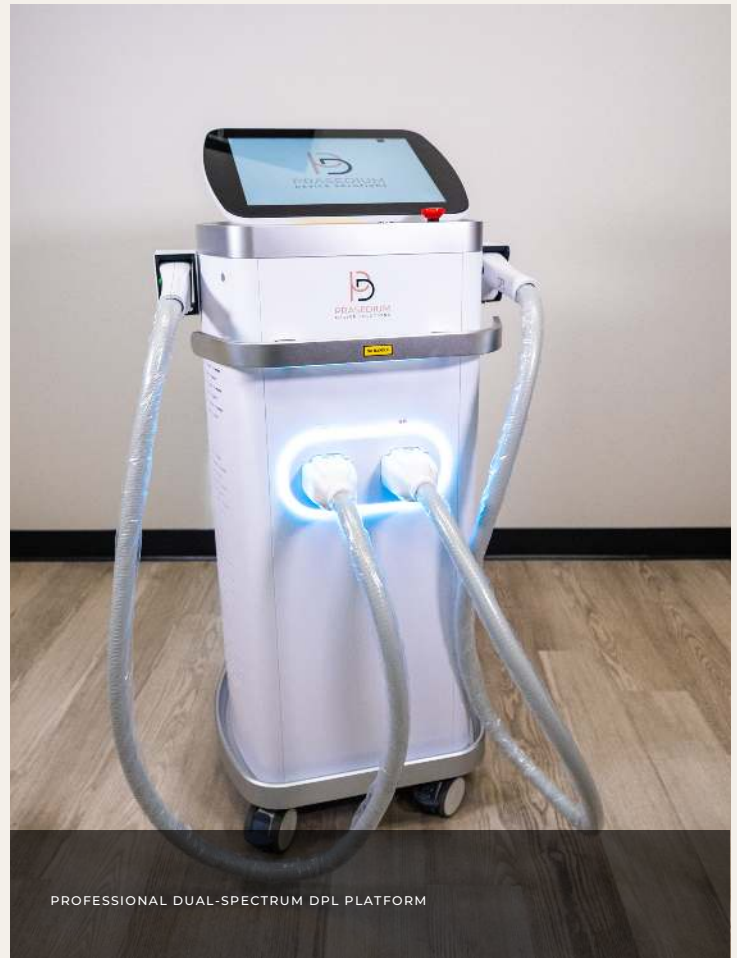

PROFESSIONAL DUAL-SPECTRUM DPL PLATFORM

4000 W

RATED OUTPUT POWER

2

TREATMENT MODES

125.7 lb

NET SYSTEM WEIGHT



PRASEDIUM
DEVICE SOLUTIONS

SELECTIVE LIGHT DELIVERY

A narrower spectrum, purposefully applied.

Manufacturer materials describe DPL as a filtered intense-pulsed-light approach designed to balance target absorption, penetration depth and controlled heat build-up.

530-750

SKIN-FOCUSED BAND

A filtered spectrum positioned for rejuvenation, pigmentation, vascular and acne-focused professional protocols.

640-750

HAIR-FOCUSED BAND

A red-to-near-infrared band positioned for melanin targeting and professional hair-reduction protocols.

OPT / AFT

EVEN PULSE DELIVERY

The pulse architecture is designed to maintain more uniform energy delivery than a sharp triangular output profile.

1-10 HZ

WORKFLOW CONTROL

Adjustable repetition supports both gliding In-Motion work and more targeted Stamp-mode delivery.

Settings, filter selection, eye protection and patient suitability must be determined by a properly trained professional using current device labelling.

BUILT FOR DAILY USE

Control at the point of care.

The system combines an angled interface, two handpieces and a mobile footprint for a practical room-to-room workflow.



01

Real sapphire contact

The handpiece window uses sapphire for direct contact during light delivery.

02

Manufacturer-rated durability

The handpiece is described by the manufacturer as rated for up to one million shots.

03

Two working modes

In-Motion and Stamp modes support different delivery techniques and treatment zones.

04

Multi-language software

The interface is designed to support guided operation across multiple language options.

PROTOCOL BREADTH

A flexible service menu.

Hair reduction	01
----------------	----

Skin rejuvenation	02
-------------------	----

Pigment-focused care	03
----------------------	----

Vascular-focused care	04
-----------------------	----

Acne-focused care	05
-------------------	----

SELECT SPECIFICATIONS

WAVELENGTHS

530-750 nm; 640-750 nm

SPOT SIZE

10 × 50 mm

SYSTEM SIZE

47 × 47 × 109 cm

NET WEIGHT

125.7 lb

Specifications are based on the supplied manufacturer brochure and should be confirmed against the exact shipping configuration.

PRACTICE POTENTIAL


One platform. Multiple service paths.

LumiClear can support a varied light-based service menu when used by trained professionals within applicable indications.


ILLUSTRATIVE PRACTICE POTENTIAL

A manageable monthly service mix.

This scenario applies the supplied prices and monthly sales assumptions consistently across each listed service or package.

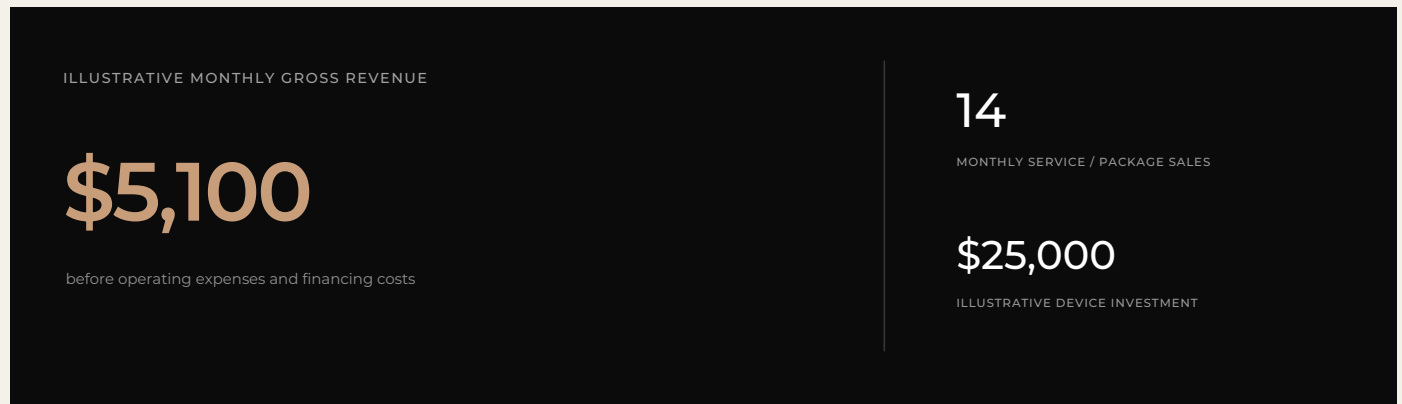

GROSS-REVENUE CONTRIBUTION

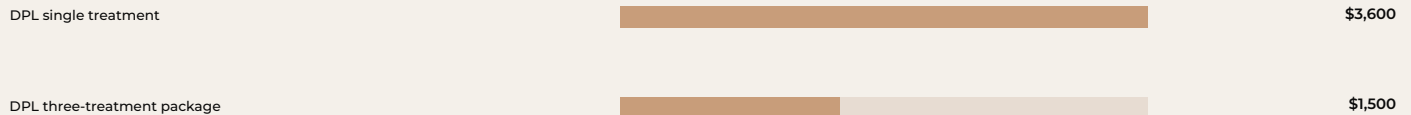

SCENARIO BOUNDARIES

Gross revenue only. Staffing, consumables, overhead, discounts, cancellations, taxes and financing costs are excluded. Actual pricing, volume and results vary.

TRANSPARENT ASSUMPTIONS

The scenario, line by line.

Each line uses the stated illustrative price multiplied by the assumed number of service or package sales in one month.

SERVICE OR PACKAGE	PRICE	SALES / MO.	GROSS REVENUE
DPL single treatment	\$300	12	\$3,600
DPL three-treatment package	\$750	2	\$1,500
ILLUSTRATIVE TOTAL		14	\$5,100

MODEL NOTE

Package counts represent package sales, not individual completed sessions. This illustration is not a forecast, guarantee of demand, or representation of profit. Practices should replace every input with their own pricing, capacity and cost assumptions.

VERSATILITY, CONSIDERED

Bring focused light into your practice.

LumiClear configuration, training and implementation support can be tailored to your practice and the protocols your team intends to provide.

CONFIGURATION • TRAINING • IMPLEMENTATION SUPPORT

